MD

# LUPA H.F

**MIMSAL**

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Lamp with circular magnifying glass.

Head protector with hinge mechanism to protect against dust and scratching.

### 1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

### 2. Caution

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- The magnifier lamp can cause fire by direct solar radiation. Always pay attention and cover the lens, particularly in areas with direct solar radiation.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

### 3. Troubleshooting and maintenance

- **Maintenance:** Only for qualified electrical engineers or other qualified personnel.
- **To change the bulb:**
  - Remove the circular light cover pressing carefully (1).
  - With a screwdriver, loosen the brackets around the fluorescent tube to remove them (2).
  - Carefully hold the fluorescent tube by the connector and gently press upward until it separates from the plug connector (3).
  - Replace the tube with a new one by following the steps above in reverse order (4).

Notify by the user / patient to MIMSAL and the competent authority of the country, any serious incident that occurs in relation to the use of the equipment

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	LUPA H.F
Light bulb	FL 22W
Diopters	5 Dp.
Diameter	12,5 cm
Electrical Protection Class	Class II
IP	20
Useful life	10.000 h
Colour temperature	6.500° K
Length of Arm	90 cm
Color	White
Weight	3 Kg
Power	220/230 V
Frequency	50/60 Hz

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