MD

MS FLEX

MIMSAL

MS FLEX
MS FLEX PLUS

cód. M183500FL
cód. M187001FL

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MS FLEX
MS FLEX PLUS

cod. M183500FL
cod. M187001FL

The design of the light fixture is highly versatile. It is suitable for medical examination and has many features that makes it ideal for use in medical centres and veterinary clinics.

The optimum working distance between the light bulb and the object being lit up is 50 cm.

1. Installation

- Requires a MIMSAL mount, which is installed before the lamp is placed in position.
- A table clamp mount is supplied and wall or ceiling mounts or a rotating base can be supplied as optional accessories.
- MIMSAL cannot guarantee the life and functionality of the product where third party accessories are used.

2. Caution

- Always disconnect the lamp from the power supply when cleaning or carrying out maintenance work.
- Clean either dry or using a weak solution of normal soap and water. Avoid using alcohol, acids or any type of organic solvent.
- Do not leave on for longer than 8 hours at a time.

3. Troubleshooting and maintenance

- **Maintenance:** Only for qualified electrical engineers or other qualified personnel.

Notify by the user / patient to MIMSAL and the competent authority of the country, any serious incident that occurs in relation to the use of the equipment

MIMSAL only accepts responsibility for maintenance, repair or changes carried out by MIMSAL staff using original MIMSAL parts.

The manufacturer reserves the right to change the characteristics of its products without prior notice.

	MS FLEX	MS FLEX PLUS
Light bulb	LED 8W	LED 12W
Useful life	50.000 h	50.000 h
Lux at 50 cm	25.000 lux	45.000 lux
Colour temperature	4.500° K	4.500° K
Colour Rendering Index	> 97%	> 97%
Electrical Protection Class	Class II	Class II
Length of Arm	70 cm	70 cm
Color	White	White
Power	110/230V	110/230V
Frequency	50/60 Hz	50/60 Hz
Dimmable	NO	YES

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