

INSTRUCTIONS FOR USE

Transport shower chair

Product code: FS 7962L




1. Characteristics

Transport shower chair is made of aluminium frame. It consists of backrest and U-shape seat made of durable PVC, flip-up armrests, folding legrest, solid rear wheels (6'') and swivel front wheels (4'') with locks.

Elements of the chair:

- a. Backrest
- b. Armrest
- c. Seat
- d. Locking pin
- e. Assembly screw
- f. Footrest
- g. Lock

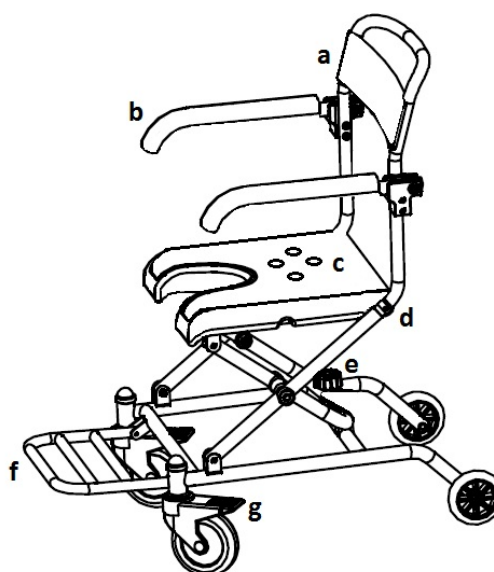

2. Application

The transport shower chair is ancillary equipment which helps the user to perform hygienic activities. The chair is recommended to people with reduced mobility, dysfunctions and motor disorders. The chair should be used only with the help of accompanying person. The chair is designed to be used inside the building at home and in hospitals.

3. Assembly

Step 1. Unfold the legrest.

Step 2. Lift the seat and lock it on the rear part of frame by tightening the assembly screw.

Step 3. Insert the tubes of the backrest into the rear part of the frame and lock the backrest with locking pins.

Step 4. Lower the armrests.

The chair is ready to use.

4. Technical data

Overall width	54 cm
Overall depth	74 cm
Seat height from the ground	47 cm
Overall height	88 cm
Seat dimensions	40 x 37.5 cm
Weight	6 kg
Maximum capacity	100 kg

5. Notes

Timago International Group does not bear responsibility of improper moving on the transport shower chair, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

Carry out the maintenance of the chair components regularly:

- The chair can be cleaned with damp cloth and cold or warm water. Difficult stains can be cleaned with a sponge. Dry away from heat sources.










7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.
Learn more at: www.timago.com. Thank you for choosing Timago!



Timago International Group
Spółka z o.o. i Spółka – Spółka komandytowa
ul. Karpacka 24/12
43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00
F.: +48 33 499 50 11
E.: info@timago.com



Rev. II, date 20.05.2021