ES Manual de instrucciones
EN Instruction manual
FR Mode d'emploi
DE Bedienungsanleitung
IT Manuale d'istruzioni

WKE008

FACIALWAXER

FUNDIDOR DE CERA DE 500 ML
500ML WAXHEATER
CHAUFFE-CIRE DE 500 ML
WACHSERHITZER MIT 500 ML
SCALDACERA 500 ML

USO DEL MANUAL

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USE OF THE MANUAL

It is important that you read and retain this user manual for future reference. It contains important information on its use and maintenance.

You can also ask a professional for guidance to make sure you are using the bed correctly.

WARNING

- Never under any circumstances attempt to open or inspect the internal components of the device. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the device in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following directives:

- Low Voltage Directive: 2014/35/EU
- Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection regarding the treatment of electrical waste.

WELKOME!

Thank you very much for your purchase. Please read carefully the instructions before using the device. We accept no responsibility for damages caused by misuse.

The company reserves the right to amend the details of the devices with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

CONTENT

On arrival of the package, check the presence of the following items:

- Wax heater

GENERAL INFORMATION

Thanks to its easy operation and perfect results, waxing beauty treatments are very popular around the world and among those professionals working in the aesthetics and hairdressing fields. They are also perfect for home use. This model is also indicated for dry and aging skins. It is used for face and body. The beauty wax used in this machine is rich in nutrient, collagen fiber and elements to remove wrinkles.

OPERATION

1. Ensure that the skin is absolutely clean before its use.
2. Plug the machine to the main supply.
3. Put the inner pot into the inner housing.
4. Turn the temperature-adjusting button clockwise. The light indicator will turn on. When the working temperature is reached, the light indicator will turn off. When the wax is melt, turn the temperature-adjusting button to the middle position again.

5. Check the wax temperature before its use.
6. Adjust the lever at 45° and spread the wax onto the skin following the direction of the hair growth. The thinner the wax layer is, the easier the hair removal becomes. Do not surpass the paper band surface. Use your hand to smooth the paper band to the direction of the hair growth. Use one hand to hold the paper band tightly and use the other one to press the band against the skin. Pull off quickly in the opposite direction of the hair growth.
7. Switch off the machine and unplug it from the main supply after use.

REMARKS AND RECCOMENDATIONS

1. Turn off the power before cleansing. To do this, switch off the power button first.
2. When you add wax, move or clean the pot, turn off the unit.
3. The machine can only be cleaned with alcohols or specialized wax removal products.
4. Do not turn on the wax machine, if the wax pot is empty.
5. Do not move the machine while it is operating.

6. After use, do not pour the wax when it is still liquid. Firstly, turn off the equipment and wait for the wax to become cold. Then, turn on the equipment again. When the wax at the bottom starts melting slightly, it will be easier to remove the wax block.

7. Pay attention not to spatter the wax on the machine, especially on the temperature controller. If this accidentally happens, clean it immediately before the wax becomes hard.

8. If the power cord is damaged, buy the component at the manufacturer or at an authorised technical service.

9. Avoid sun exposure the 24 hours following hair removal.

10. Use the device responsibly and take safety precautions.

11. Keep the device away from children.

TECHNICAL SPECIFICATIONS

CAPACITY	500 ML
INNER POT DIAMETER	110 MM
TEMPERATURE	95°C
RATED POWER	100 w
RATED VOLTAGE	AC 220V ~ 240V

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